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PATIENT: male, 2 months old.

We present the case of L. B. a newborn male affected by Pierre Robin syndrome: who presented feeding difficulties and a mild to severe upper airway obstruction.

Feeding difficulties was overcome by insertion of a nasogastric tube and a nasopharyngeal prong was used to bypass the upper airway obstruction.


As soon as L. B. Was 2 months old, we approached the mandibular bone lengthening technique using the Brevi-Sesenna internal distraction device.

Distraction started 24 hours after the surgical procedure at the rate of 1 mm a day. Within eight days L. B. Was able to breath without the nasopharyngeal prong.

Distraction was stopped once the device reached maximum length with a mandibular lengthening of 15 mm. L. B. Was then feed p. o. And on five days time reached regular diet regimen.

Preoperative view of L. B. with the nasopharyngeal prong used to bypass the pharyngeal obstruction;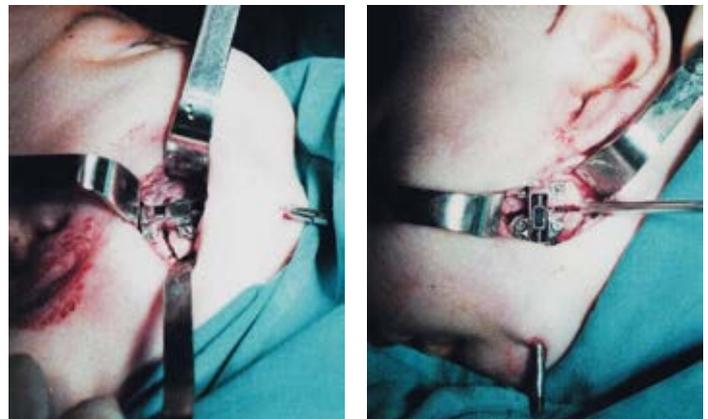

Brevi-Sesenna internal device placed in angular region with an anterior-posterior vector;

Postoperative view when distraction was stopped (mandibular lengthening of 15 mm);