DESCRIZIONE:

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DESCRIPTION:

Alveolar distractors are devices which, with intraoral application, allow to increase progressively the vertical growth of alveolar region. Ridge regeneration surgery. Distractors can be customized according to the specific dimensional requirements; 20mm is standard distraction length.

MATERIAL:

Alveolar distractors: stainless steel AISI 316L.. The composition _ Fe, Cr, W, Mo is referenced by ASTM F138.
Fixation screw: titanium grade 5 A.S.T.M. F136 diameter 1,6 mm and a length varying from 4 to 13 mm.

THE MATERIALS DESCRIBED ABOVE IS AVAILABLE DECONTAMINATED, NON-STERILE, DISPOSABLE, NOT 'REUSABLE.

INTENDED USE:

Exclusively for vertical increase of alveolar region.

CONTRAINDICATIONS:

- Allergic to any of the titanium and stainless steel.
- Special conditions of the patient, senility, alcoholism, infections, altered bone metabolism and generally systemic diseases, endocrine, metabolic and immune system.
- Re-use.

The device is disposable and if it is reused involves the following major issues:

- INFECTION: also subsequent manipulation does not guarantee a microbial conforming to the type of sterilization. Pyrogenicity.
- STRAIN: the device could suffer in the subsequent handling of small deformations which would make it unsuitable for use.

PRECAUTIONS:

Surgeon should not begin a clinical use of the distractor, screws and drills without having previously read the instructions for use, must also use materials and related equipment in specialized environment (clinic or operating room). The user must check the integrity of the packaging before using the product and after sterilization.

The manufacturer assumes no responsibility for improper use.

Use with the included screws and screwdriver.

Do not use the product whose packaging is damaged.

STERILIZATION:

The production of alveolar distractors is carried out with great care to ensure the safety and quality of the surgeon's operative result. Medical staff must contribute to the achievement of laying the proper care in the handling and use of this device. For mandibular distractors there are no particular restrictions into the method of sterilization, Cizeta Surgical recommends autoclaving, vacuum, at a temperature of 134 ° C (273 ° F) for a duration of 10 minutes.

The device can be sterilized with ethylene oxide. The sterilization process must be validated in accordance with the UNI EN ISO 11135-1:2008. Warning: the residue of ethylene oxide, if non-compliant, it can cause serious infections. Please note that the sterilization process chosen must still be able to ensure compliance with the standard EN556, which states that the theoretical probability that the device is present viable micro-organisms must be less than or equal to 1 in 10⁶ (SAL = 10⁻⁶, where for SAL means Sterility Assurance Level). It is responsibility of the user to validate all phases of sterilization and verify the sterility for all products and at all stages used.

The devices, which are supplied **unsterilized**, are packaged in clean room bags with a side medical grade paper and a plastic transparent polypropylene / polyester. On the package there are two chemical indicators:

- for steam dye toning to brown color change
- for ethylene oxide dye toning to yellow color change

INSTRUCTIONS FOR USE:

Alveolar distractors are placed intraoral. The seat of the insertion is selected appropriately depending on number and size of the alveolar and of the need to increase vertically. The distractors are technically designed so as to obtain, in the osteogenesis, the desired direction of distraction vector. It should be left to escape the distractor activation framework from the mucosa. Activation of the distractor can be achieved by inserting the appropriate key in the framework of the rod and is rotated in a clockwise direction (as indicated by the arrow on the stick of reference). The "timing" of distraction should be appropriate to the age and tolerance of the patient, the extent of mandibular elongation and type of existing bone structure. Once distraction process is completed the activation rod can be removed from the distractor, leaving the distraction body under mucosa. When reached the desired distraction, need to wait for bone consolidation is completed (6-8 weeks) before removing the device via the same route.

Antibiotic prophylaxis is recommended for 5 days.